

- Clean all stains as soon as possible. Note: **IC+ can be cleaned with any healthcare detergent or disinfectant** including Virox 5, CaviCide, Rescue, WexCide and bleach when used in the quantities and manners specified by the cleaning agents' labeled cleaning instructions.
- Specifically formulated for the healthcare industry, IC+ is the only upholstery option available that is warranted against tears, cracking and punctures. IC+ is warranted against tears, cracking and punctures for **10 years**.

**Note:** For working areas that frequently use items that can stain IC+ (i.e., pen), it is recommended to specify darker colors. (Please refer to the "IC+ Stain Removal Rating" for more information on products that stain IC+ and the level of staining.)

## IC+ CLEANING INSTRUCTIONS

STAIN TYPE	Recommended Cleaner(s)	Cleaning Instructions
ADHESIVE TAPE	<ul style="list-style-type: none"> <li>• Mild soap and water</li> <li>• Goof Off</li> <li>• Windex</li> <li>• Goo Gone</li> <li>• Clorox Wipe</li> <li>• 10:1 bleach</li> </ul>	<ul style="list-style-type: none"> <li>• Remove adhesive tape as soon as possible.</li> <li>• Use recommended cleaners in the quantities and manners specified by the cleaning agents' labeled cleaning instructions.</li> <li>• If the stain is left over a 24 hour period, clean surface as directed using recommended cleaners to remove as much of the stain as possible. Goo Gone can be used to remove any remaining tape residue.</li> </ul>
BLOOD	<ul style="list-style-type: none"> <li>• Mild soap and water</li> <li>• Any healthcare detergent or disinfectant including Virox 5, Cavicide, Rescue, Wexcide and bleach</li> </ul>	<ul style="list-style-type: none"> <li>• Use recommended cleaners in the quantities and manners specified by the cleaning agents' labeled cleaning instructions.</li> <li>• If the stain is left over a 24 hour period, clean surface as directed using recommended cleaners to remove as much of the stain as possible.</li> </ul>
COFFEE	<ul style="list-style-type: none"> <li>• Mild soap and water</li> <li>• Any healthcare detergent or disinfectant including Virox 5, Cavicide, Rescue, Wexcide and bleach</li> </ul>	<ul style="list-style-type: none"> <li>• Coffee stains should be <b>cleaned immediately</b> to minimize staining of surface using recommended cleaners in the quantities and manners specified by the cleaning agents' labeled cleaning instructions.</li> <li>• Darker colors of IC+ (black) will not show or will minimize the stain if it is cleaned immediately.</li> <li>• For stains on lighter colors of IC+ that are left over a 24 hour period, clean surface using an Isopropanol-based cleaner such as Cavicide or a Phenol-based cleaner such as Wexcide for best results. A slight stain will remain on the surface.</li> </ul>
DIET COLA, HAND CREAM, KETCHUP, MILK, SOY SAUCE, WINE	<ul style="list-style-type: none"> <li>• Mild soap and water</li> <li>• Any healthcare detergent or disinfectant including Virox 5, Cavicide, Rescue, Wexcide and bleach</li> </ul>	<ul style="list-style-type: none"> <li>• Use recommended cleaners in the quantities and manners specified by the cleaning agents' labeled cleaning instructions.</li> <li>• Stain will be completely removed.</li> </ul>

**HAND SANITIZER**

- Mild soap and water
- Any healthcare detergent or disinfectant including Virox 5, Cavicide, Rescue, Wexcide and bleach
- Clean stain as soon as possible using recommended cleaners in the quantities and manners specified by the cleaning agents' labeled cleaning instructions.
- Darker colors of IC+ will show a slight discoloration, which can be further reduced using hot water and mild soap. Stains left over a 24 hour period may require more scrubbing for the stain to be reduced.
- Lighter colors will not show or will minimize surface discoloration.

**IODINE**

- Hydrogen peroxide-based products only
- Iodine can only be removed (with usually a slight stain remaining) with hydrogen peroxide-based products and before iodine has completely dried.
- Iodine stains should be **cleaned immediately** to minimize staining of surface using hydrogen peroxide-based cleaners in the quantities and manners specified by the cleaning agents' labeled cleaning instructions. The longer iodine has been left on the surface, the darker the stain will be.
- It is recommended to specify darker IC+ colors for areas where iodine is often used.

**LIPSTICK**

- Mild soap and water
- Bleach
- Vinegar (full strength)
- Goo Gone
- Clean stain as soon as possible using recommended cleaners in the quantities and manners specified by the cleaning agents' labeled cleaning instructions.
- Complete stain removal is dependent on the severity of the stain.
- Light markings of lipstick can usually be completely removed.
- Goo Gone can be used to remove heavier lipstick stains. A slight color film may be left on the surface from heavier markings of darker colors (i.e., red) after cleaning.

**MARKER PEN**

- Isopropanol-based cleaner or a phenol-based cleaner may reduce the severity of the stain.
- It is recommended to specify darker colors in areas where markers/pens are often used.
- Cleaning the surface using an Isopropanol-based cleaner such as Cavicide or a Phenol-based cleaner such as Wexcide may reduce the severity of the stain (some fading).

**OTHER SURFACES CLEANING INSTRUCTIONS**

**FABRICS**

- Keep clean with regular vacuuming. For spills, absorb as much as possible with a damp cloth, but do not rub. For stains, we recommend professional cleaning.

**VINYL**

- Clean with soft cloth dampened with mild soap solution. For stains, a sponge or soft brush may be used with mild non-abrasive cleaner.

**METAL/LAMINATE**

- Dust regularly with a soft, clean cloth. Occasionally, it may be necessary to use a mild non-abrasive cleaner.

**URETHANE**

- Wash with mild soap and water. Do not use any solvents, alcohol or stain removers.

**CHAIR SHELLS/FRAMES**

- Wipe with a damp cloth.